# MORSA

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**MORSA: The international competence for dental waxes**

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*MORSA dental waxes have permanently been improved and tested in cooperation with numerous dentists and experts in the field of dental techniques. The MORSA management welcomes every practical hint of further improvements and puts a premium on every accepted suggestion.*

*Our representative in Germany: Müller Dental GmbH, Postfach 11 30, 51779 Lindlar, Tel. 0 22 66 / 47 42 - 0, Fax 0 22 66 / 34 17 .*

*Internet: [www.mueller-omicron.de](http://www.mueller-omicron.de) · E-Mail: [info@mueller-omicron.de](mailto:info@mueller-omicron.de)*

**MORSA** Wachswarenfabrik Sallinger GmbH  
Nordstraße 3 · 86381 Krumbach / Germany  
Telefon 0 82 82 / 40 95 / 96 / 97 · Fax 0 82 82 / 58 96  
Internet: [www.morsa.de](http://www.morsa.de) · E-Mail: [morsa@t-online.de](mailto:morsa@t-online.de)

# MORSA

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The international competence for dental waxes

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The tradition of the Sallinger company dates back to the year 1647, documented in historically verifiable sources. From the centuries-old family tradition, the **MORSA** Wachswarenfabrik Sallinger GmbH company developed in a dynamic manner into one of the best manufacturers of dental waxes. Meanwhile **MORSA** dental products are not only widely known in Germany but also in other European countries as well as overseas.

**MORSA** dental waxes are constantly improved. The company is able to offer a wide range of dental waxes, which have been developed and tested in cooperation with many dentists and experts in the field of dental techniques.

As several **MORSA** quality waxes are sold under private label, you will not always recognize **MORSA** waxes by their labels. If you have some doubts about the authenticity of **MORSA** products, you are welcome to consult us. Then we can determine for you which are original **MORSA** waxes and which not. We are also very flexible with regard to special developments and we are able to produce individual wax qualities for you.

Should you require references from renowned European dental companies, we could send you the respective literature with pleasure.

We will gladly help you with technical questions and further information and remain

Yours faithfully  
Dr. Hermann Sallinger  
**MORSA** Wachswarenfabrik Sallinger GmbH

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## 1.1 Pink

### wax sheets:

Supple, firm edges, harden rapidly, can be cut precisely, easily sculptured.

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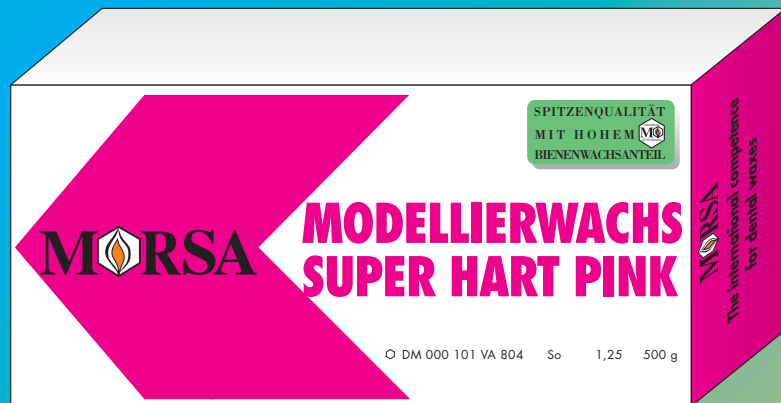
#### 1.1.1 "Extra hard Super"

The perfect summer wax – pleasantly supple during use, minimal contraction during cooling and hard at room temperature.

Best.Nr. DM 000 101 VA 804      175 x 80 x 1,25 mm      brutto ca. 500 g / gross weight: appr. 500 g

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#### 1.1.2 "Super hard pink"

A perfect, new development in sheet waxes. Optimum sculpturing properties, cools rapidly, minimal shrinkage, very hard! The indispensable setting-up wax for use in summer.

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## 3.1 Waxes for chrome cobalt - sculpturing waxes

Perfect for joining all waxes and prefabricated resin patterns when waxing-up patterns for chrome cobalt appliances.

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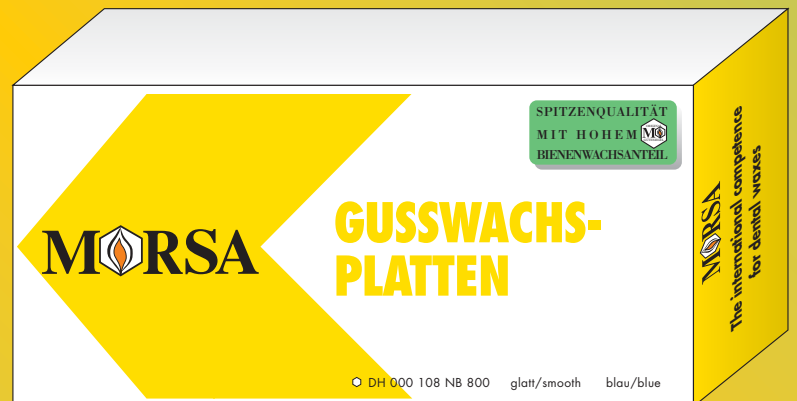
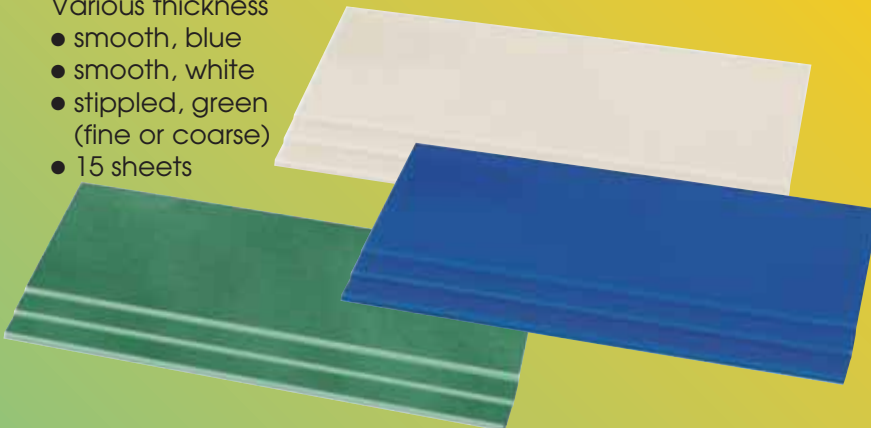
## 2.6 Aesthetic wax

For learning and designing the natural contours and shades of wax patterns. *Shade system* available in shades white to brown (four shades). Approximately 25 g tubes (four tubes in a metal box). *Re-fill box:* Single tubes for placing in the box.

Best.-Nr.	4er Set	
DR 000 122 HD 816	elfenbein/ivory	50 g Dose/50 g tube
DR 000 123 HD 809	creme/cream	50 g Dose/50 g tube
DR 000 124 HD 810	leicht bräunlich/light brown	50 g Dose/50 g tube
DR 000 125 HD 803	bräunlich/brown	50 g Dose/50 g tube
DR 000 126 HD 801		

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## 2.7 Dipping waxes

Perfect precision of fit, dimensionally stable, will not drip, harden rapidly, minimal contraction.  
 • yellow • red

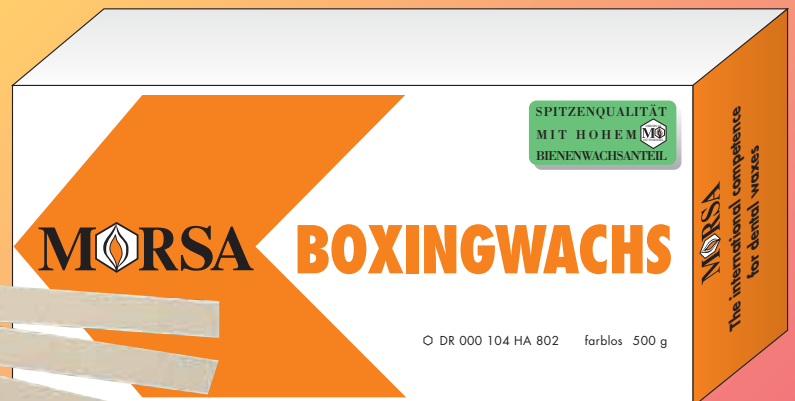
Best.-Nr. DR 000 127 HD 803	gelb/yellow	80 g Dose/80 g tube
Best.-Nr. DR 000 128 HD 805	rot /red	80 g Dose/80 g tube

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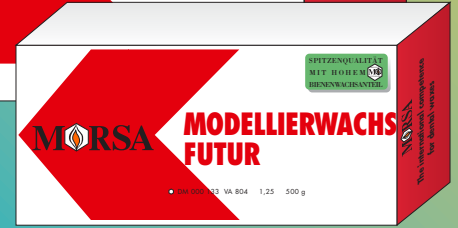
*Practical hint from Herr Miller (Master Dental Technician): "FUTUR" wax sheets can be optimally adapted in the mouth due to colour and stability.*

Best.-Nr. DM 000 133 VA 804 175 x 80 x 1,25 mm  
Best.-Nr. DM 000 134 VA 804 175 x 80 x 1,25 mm

brutto ca. 500 g/gross weight: appr. 500 g  
brutto ca. 2,5 kg/gross weight: appr. 2,5 kg

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## 1.1.4 "2000" wax sheets

All-round wax for all applications in  
prosthetic departments.

Best.-Nr. DM 000 121 VA 804 175 x 80 x 1,25 mm  
Best.-Nr. DM 000 122 VA 804 175 x 80 x 1,25 mm

brutto ca. 500 g/gross weight: appr. 500 g  
brutto ca. 2,5 kg/gross weight: appr. 2,5 kg

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## 1.1.5 "Euro" wax sheets

Universal wax for all summer and  
winter temperatures (except ex-  
tremely high temperatures).  
An alternative wax for a  
multitude of applications  
in Europe.

Best.-Nr. DM 000 123 VA 804 175 x 80 x 1,25 mm  
Best.-Nr. DM 000 124 VA 804 175 x 80 x 1,25 mm

brutto ca. 500 g/gross weight: appr. 500 g  
brutto ca. 2,5 kg/gross weight: appr. 2,5 kg

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## 1.1.6 "Top-Techno"

The allround sheet wax for all acrylic denture procedures. For normal summer and winter temperatures.

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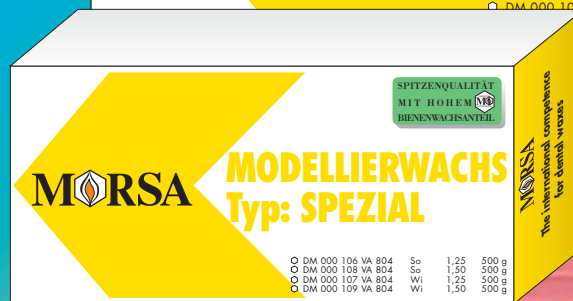
## 1.1.7 "Special"

Our best-selling sheet wax. The "all-round setting-up wax" for normal repairs in summer and winter. Modelling wax for all acrylic denture procedures – good sculpturing properties, high resistance to fracture, minimal thermal contraction.

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- blau

### 2.3 Milling wax

Perfect for milling. Machines well to create smooth surfaces.

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- rot
- blau

### 2.4 Cervical wax

For precision fit crown margins. Applied non-stressed, minimal solidification shrinkage – perfect for crown margins. Adapts well.

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**Practical hint from Herr Miller (Master Dental Technician):** *Blue cervical wax is perfect for margins marked with a red marker – the cervical margin remains visible.*

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- violett

### 2.5 Blocking-out wax

Special wax for blocking out cavities and rough surfaces – high melting point, extremely hard and easily wetted with die spacer.

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MORSA sculpturing waxes – fully tested in various laboratories, proven in everyday use, specifically designed to meet the needs of crown and bridge technology.

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## 2.1 Crown waxes

All-round crown wax for the demanding precious metal technician – made of only the best raw materials, unique properties for both building-up and carving.

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- grün
- mittelgrau

## 2.2 Inlay waxes

The perfect inlay wax, minimal contraction, optimum firmness and good sculpturing properties.

- green
- median grey

Best.-Nr. DR 000 116 HD 813  
Best.-Nr. DR 000 117 HD 814

grün / green  
mittelgrau / median grey

50 g Dose / 50 g tube  
50 g Dose / 50 g tube

# 1. PROTHETIK/PROSTHETICS

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### 1.2.1 Bißwallwachs in Stangen

- rosa – normale Härte
- gelb – extra weich

## 1.2 Additional waxes for prosthetics:

### 1.2.1 Wax bite-blocks – rods

- pink – standard hardness
- yellow – extra soft

Best.-Nr. DB 000 105 AU 803	gelb/yellow	180 x 10 x 10 mm	brutto ca. 2 kg/gross weight: appr. 2 kg
Best.-Nr. DB 000 106 AU 804	rosa/pink	180 x 10 x 10 mm	brutto ca. 2 kg/gross weight: appr. 2 kg

### 1.2.2 Wachsbißwälle vorgeformt

- rosa – normale Härte
- gelb – extra weich

### 1.2.2 Wax bite-blocks – preformed

- pink – standard hardness
- yellow – extra soft

Best.-Nr. DB 000 112 AU 803	klein/small	gelb/yellow	69 Stück/69 pieces
Best.-Nr. DB 000 113 AU 803	groß/large	gelb/yellow	69 Stück/69 pieces
Best.-Nr. DB 000 114 AU 804	klein/small	rosa/pink	69 Stück/69 pieces
Best.-Nr. DB 000 115 AU 804	groß/large	rosa/pink	69 Stück/69 pieces

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## 1.2.3 Wax bite-blocks with flavouring

- green – apple
- red – strawberry

Bite-taking with aromatic bite-blocks helps prevent the patient gagging.

Best.-Nr. DB 000 116 AU 813  
 Best.-Nr. DB 000 117 AU 813  
 Best.-Nr. DB 000 118 AU 805  
 Best.-Nr. DB 000 119 AU 805

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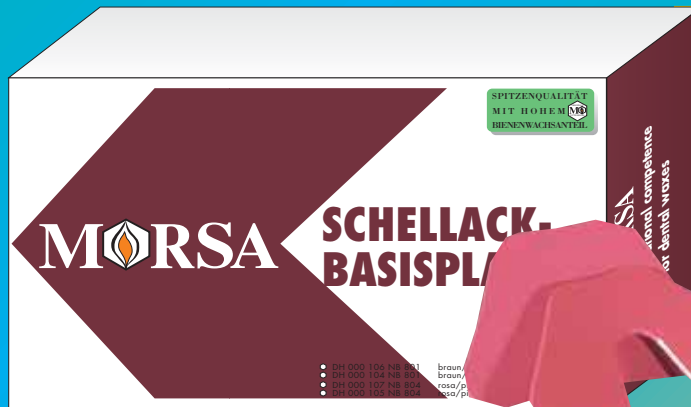
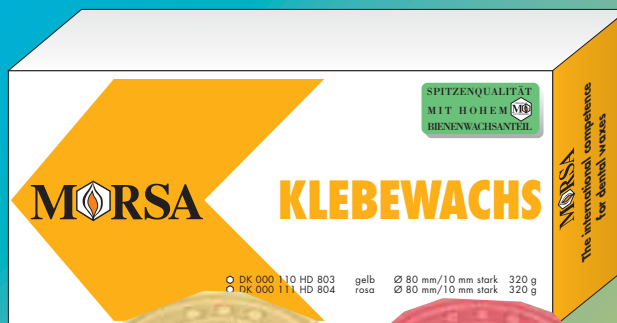
## 1.2.5 MORSA sticky wax – 8 sheets

- yellow
- pink

Highly adhesive, hard but not brittle, supple.

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
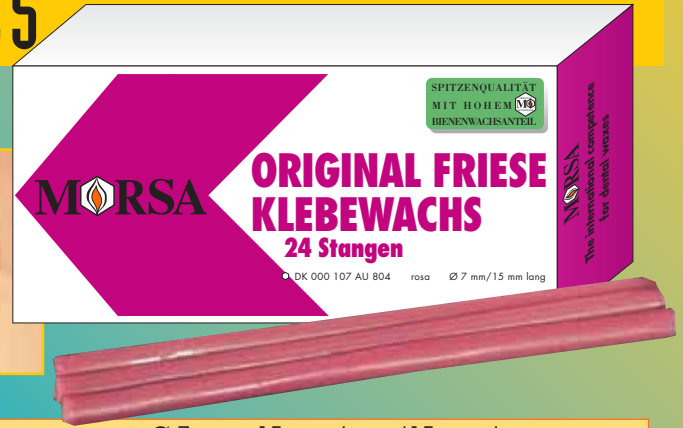
## 1.2.6 Original Friese sticky wax

- pink 24 sticks
- Exclusive Friese speciality

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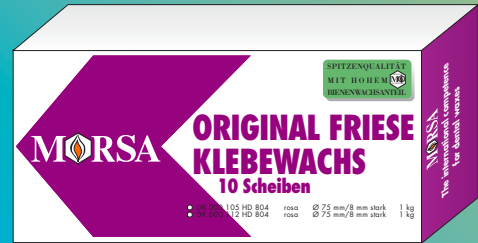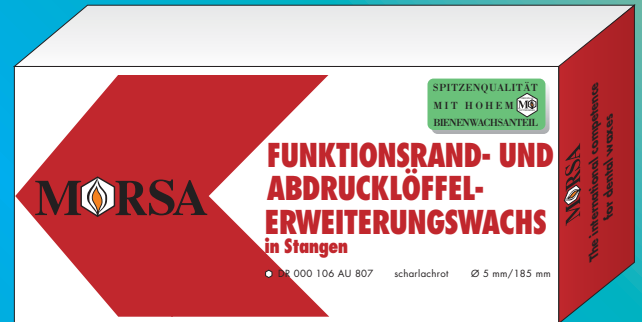
- 32 Stangen

## 1.2.7 Functional periphery wax and impression tray extension wax

Proven over decades, highly plastic, exact dorsal postdams, does not need warming.

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## 1.2.8 Alu bite-wax

Records indentations precisely and is easily cut.

Best.-Nr. DB 000 104 HA 806 silber / silver 175 x 80 x 2 mm brutto ca. 250 g / gross weight: appr. 250 g